




BIOPTRON[®] 
HYPERLIGHT THERAPY SYSTEM By Zepter Group

F I L T E R S

E N G L I S H (E N)

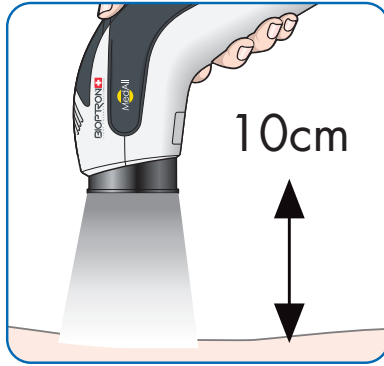

1



2



3



4

HOW TO USE?

From a distance of 10 cm, position the BIOPTRON device with the desired filter to cover the area you wish (Picture 1) to treat. When treating areas larger than the color filter diameter, treat them one by one according to the recommended time.

HOW TO CHANGE THE FILTER?

Carefully and without force take out the filter (Picture 2) to be replaced and insert the desired color filter as indicated in Picture 3. Hold the outer ring of the filter rather than the lens.

HOW TO CLEAN THE FILTER? (Picture 4)

1. Remove the filter from the device by holding the outer ring of the filter.
2. Use a soft, wet, lint-free cloth to avoid scratching the glass surface.
3. Dry the filter with another lint-free cloth.

WARRANTY

- 5 years if used as intended and if warnings are respected
- This warranty covers material and manufacturing defects
- This warranty does not cover aesthetic changes resulting from ordinary wear and tear that do not affect product performance, or damages caused by product misuse.

WARNINGS

1. Do not use any chemicals (e.g. isopropyl alcohol) to clean the inner or outer side of the glass surface and filter ring.
2. Store the filter in a protective pouch to prevent the accumulation of dust and discoloration due to exposure to sunlight.
3. The filter is made of fragile glass, so it must be handled with care!
4. User is not allowed to remove filter lens.



PMD-ME145-22-EN
BIOPTRON FILTER IFU



BIOPTRON AG
SIHLEGGSTRASSE 23, CH-8832 WOLLERAU, SWITZERLAND
PHONE: +41 43 888 28 00 | E-MAIL: LIGHT@BIOPTRON.COM



WWW.BIOPTRON.COM

ENGLISH (EN)